

# Case Study for Abdominal Soft Tissue Coagulation

---

**Prepared by:** Felicia A. Whitney R.N.

**Date:** 04-14-10

Disclaimer: The contents of this publication are provided as a convenience only and is not intended to replace sound judgment or serve as the standard of care. Palomar's suggested guidelines are set forth in the operator's manuals and clinical treatment guides for Palomar products. Ultimately, a practitioner is solely responsible for the care of a particular client in light of all circumstances presented by the client, the diagnostic and treatment options at hand, and the available resources.

# Profile

---

<b>Case Study for Abdominal Soft Tissue Coagulation By: Khalil Khatri, MD</b>	
Physician name	Khalil Khatri, MD
Address	15 Network Drive
City, state, ZIP Code	Burlington, MA 01803
Phone number	781-993-2300
Fax number	781-993-2330
Web site address	<a href="http://www.palomarmedical.com">www.palomarmedical.com</a>
Contact name	Felicia A. Whitney
Title	R.N.
Phone number	781-993-2300
Fax number	781-993-2330
E-mail address	<a href="mailto:fwhitney@palomarmedical.com">fwhitney@palomarmedical.com</a>

Disclaimer: The contents of this publication are provided as a convenience only and is not intended to replace sound judgment or serve as the standard of care. Palomar's suggested guidelines are set forth in the operator's manuals and clinical treatment guides for Palomar products. Ultimately, a practitioner is solely responsible for the care of a particular client in light of all circumstances presented by the client, the diagnostic and treatment options at hand, and the available resources.

**Case Study for Abdominal Soft Tissue Coagulation By: Khalil Khatri, MD**

**Introduction:**

Patient presented with excess sagging skin tissue in the mid-abdominal area.

**Discussion:**

Patient desired a non-invasive procedure to improve the appearance of the sagging abdominal skin.

**Treatment with StarLux® 500 System:**

Single treatment with the Palomar® LuxDeepIR™

Right side treated with 88 J/cm<sup>2</sup>, 5 seconds

Left side treated with 116 J/cm<sup>2</sup>, 10 seconds

**Conclusion:**

No significant difference noted between results using different treatment settings on either side of the abdomen. There was significant improvement on both sides treated with the LuxDeepIR™.

**Patient Quote:** "I can't believe I can wear a 2-piece bathing suit again!"

Palomar and StarLux are registered trademarks and LuxDeepIR is a trademark of Palomar Medical Technologies, Inc. (Burlington, MA)

