

Case Study for Facial and Neck Soft Tissue Coagulation

Prepared by: Felicia A. Whitney R.N.

Date: 04-13-10

Disclaimer: The contents of this publication are provided as a convenience only and is not intended to replace sound judgment or serve as the standard of care. Palomar's suggested guidelines are set forth in the operator's manuals and clinical treatment guides for Palomar products. Ultimately, a practitioner is solely responsible for the care of a particular client in light of all circumstances presented by the client, the diagnostic and treatment options at hand, and the available resources.

Profile

Case Study for Facial and Neck Soft Tissue Coagulation By: Khalil Khatri, MD	
Physician name	Khalil Khatri, MD
Address	15 Network Drive
City, state, ZIP Code	Burlington, MA 01803
Phone number	781-993-2300
Fax number	781-993-2330
Web site address	www.palomarmedical.com
Contact name	Felicia A. Whitney
Title	R.N.
Phone number	781-993-2300
Fax number	781-993-2330
E-mail address	Fwhitney@palomarmedical.com

Disclaimer: The contents of this publication are provided as a convenience only and is not intended to replace sound judgment or serve as the standard of care. Palomar's suggested guidelines are set forth in the operator's manuals and clinical treatment guides for Palomar products. Ultimately, a practitioner is solely responsible for the care of a particular client in light of all circumstances presented by the client, the diagnostic and treatment options at hand, and the available resources.

Case Study for Facial and Neck Soft Tissue Coagulation By: Khalil Khatri, MD

Introduction:

Patient presented with loose, sagging skin in neck and lower facial tissue.

Discussion:

Patient desired non-surgical approach to improve the appearance of the skin in the neck and lower facial areas.

Treatment with StarLux® 500 System:

Bilateral neck and lower facial areas treated in single treatment with LuxDeepIR™

Right side treated with 78 J/cm² 5 seconds


Left side treated with 106 J/cm² 10 seconds


Conclusion:

Although different settings were used for each side of the neck and face, there was no notable difference in results and there was significant improvement after one treatment.

Patient Quote: "It looks wonderful! I can't believe the difference with just one treatment."

Palomar and StarLux are registered trademarks and LuxDeepIR is a trademark of Palomar Medical Technologies, Inc. (Burlington, MA)

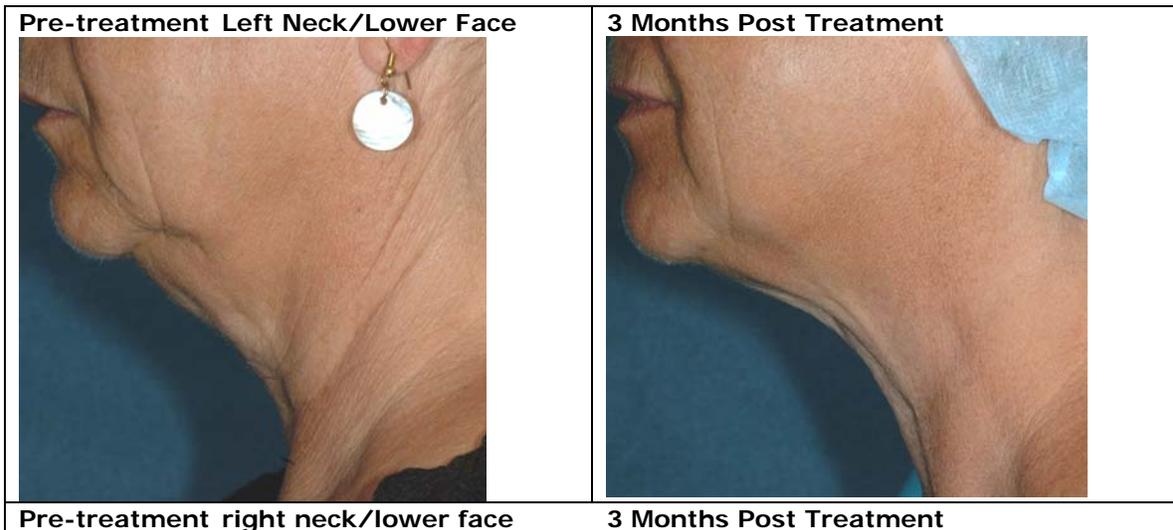

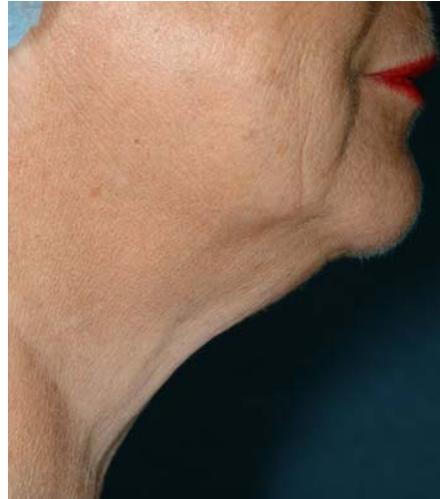

Case Study for Facial and Neck Soft Tissue Coagulation By: Khalil Khatri, MD

Pre-treatment:

3 Months Post Treatment



Disclaimer: The contents of this publication are provided as a convenience only and is not intended to replace sound judgment or serve as the standard of care. Palomar's suggested guidelines are set forth in the operator's manuals and clinical treatment guides for Palomar products. Ultimately, a practitioner is solely responsible for the care of a particular client in light of all circumstances presented by the client, the diagnostic and treatment options at hand, and the available resources.